Guide for Use


Infyna Chic[™]
Intermittent Catheter

Continence Care

- 1** Gather all equipment and prepare the area.
- 2** Wash your hands as advised by your healthcare practitioner.
- 3** Position yourself to carry out intermittent catheterization. Manipulate your clothing to access your genitalia.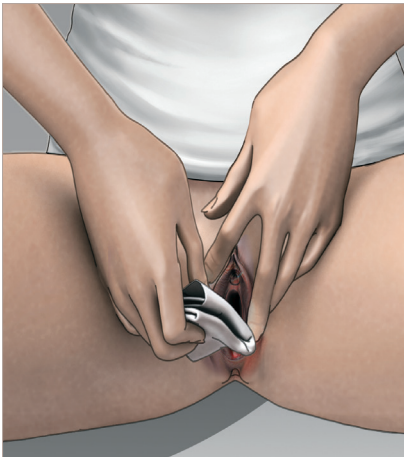

- 4** Spread the labia with two fingers of your non-dominant hand. If advised by your healthcare practitioner, cleanse around the area of the urethral opening and genitalia with your dominant hand, as you were trained.

- 5** Open the case by flipping up the cap, with your dominant hand, which breaks the seal. Leave the product in the case. **NOTE:** if seal is already broken, do not use the catheter.

If you are using a collection bag accessory (not included), attach to the funnel end of the catheter. No need to remove catheter from the case.

- 6** Hold the case with your non-dominant hand and remove catheter from the case with your dominant hand, touching only the funnel.

- 7** Part the labia with two fingers of your non-dominant hand, and insert the catheter with your dominant hand into the urethral opening, until urine starts to flow.

- 8** Once urine flow has stopped slowly withdraw the catheter. If urine flows again, pause until urine stops. After you're sure the bladder is empty, remove the catheter completely.

9 To dispose, place catheter back inside case and close the cap. You may dispose in a waste bin immediately or later when it is more convenient. **DO NOT FLUSH DOWN TOILET.**

10 Wash your hands.

Infyna Chic™

Intermittent Catheter

Watch the instructional video:
**[www.Hollister.com/
InfynaChicVideo](http://www.Hollister.com/InfynaChicVideo)**

Description

This product is a hydrophilic, single-use intermittent catheter that is ready to use after you remove it from the case. This intermittent catheter features two smooth eyelets as well as a color- (colour-) coded funnel. Not made with phthalates. Not made with natural rubber latex.

Indications

This intermittent catheter is a flexible tubular device that is inserted through the urethra by female patients who need to drain urine from the bladder.

WARNINGS

To help reduce the potential for infection and/or other complications, do not reuse.

If discomfort or any sign of trauma occurs, discontinue use immediately and consult your healthcare practitioner.

PRECAUTION: Please consult your healthcare practitioner before using this product if any of the following conditions are present:

- Severed urethra
- Unexplained urethral bleeding
- Pronounced stricture
- False passage
- Urethritis – inflammation of the urethra

PRECAUTION: Self-catheterization (catheterisation) should follow the plan of care and advice given by your healthcare practitioner and be carried out only in accordance with the instructions for use provided. Because catheterization (catheterisation) frequency varies by person, the recommended frequency of your catheterization (catheterisation) should be provided by your healthcare practitioner. For any other questions about your catheterization (catheterisation), please contact your healthcare practitioner.

NOTE: Store boxes in a flat position and at normal room temperature.

Do not use if package is damaged.

This product consists of: 5 in (13 cm) hydrophilic-coated PVC catheter and funnel (not made with phthalates), contained within a polypropylene and polyethylene case.

USA: Rx Only.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician or other healthcare practitioner licensed under state law to order this product.

Hollister Incorporated

2000 Hollister Drive

Libertyville, Illinois 60048

1.800.323.4060

www.hollister.com

The Hollister logo and Infyna Chic are trademarks of Hollister Incorporated.

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© 2021 Hollister Incorporated.