At Hollister, we are always working to make our products better. The features we have added to the Lock 'n Roll Microseal Closure are based on feedback from users like you.

Transparent Flap

For enhanced security, and to protect the closure from accidentally being pulled open

Flexible Ribbed Strips

To facilitate ease of opening, draining, cleaning, and closing

Curved Closure

Designed to rest comfortably against your body

Before you apply your new pouch, close the pouch tail (see fig 6 and fig 7)

Opening

(fig 1)

- Lift from the center of the transparent flap (fig 1)

Draining

(fig 2)

(fig 3)

- Position fingers on both sides of the pouch tail, between the ribbed strips, and squeeze from the sides (fig 2)
- Continue to hold the pouch open for cleaning (fig 3)

Cleaning

(fig 4)


(fig 5)

- Clean and dry the inside of the pouch tail (fig 4)
- Wipe the outside edge of the pouch tail (fig 5)

Closing

(fig 6)

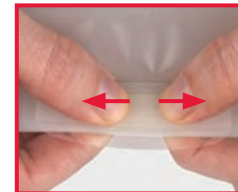(fig 7)

- Roll the pouch tail tightly
- Extend the transparent flap as far as it will reach when closing (fig 6)
- To close, start at the center and press. Continue pressing to the outer edges. You will feel the fasteners seal together (fig 7)

* For more details, please see the enclosed **Instructions for Use.**

Hollister Incorporated
Libertyville, IL USA
1.888.740.8999

Distributed by
Hollister Limited
Aurora, Canada
1.800.263.7400
www.hollister.com

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*Before you apply your new pouch, close the pouch tail
(see fig 6 and fig 7)*

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