





# The CeraPlus™ Skin Barrier Product Evaluation CLINICIANS' FEEDBACK

Based on 193 patient evaluations completed by WOC Nurses in the United States.

PATIENTS' AGE

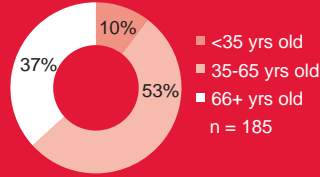

TYPE OF OSTOMY

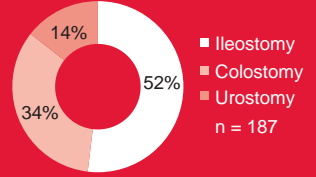

More than 100 WOC Nurses recently evaluated the new CeraPlus skin barrier with Remois Technology<sup>1</sup> with 193 ostomy patients<sup>2</sup>. We're excited to report that they have observed improvement of the peristomal skin condition and average wear time with the CeraPlus skin barrier.




**87%** n = 188  
**OVERALL SATISFACTION**  
 with the CeraPlus skin barrier for the patient

**82%** n = 182  
**LIKELIHOOD TO CONTINUE TO USE & RECOMMEND**  
 the CeraPlus skin barrier for the patient

## Improvement of Skin Condition

**Before Using**  
 the CeraPlus skin barrier\*

■ Some issues with peristomal skin  
 ■ No peristomal skin issue  
 n = 176

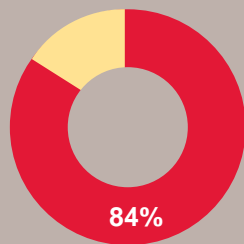

**After Using**  
 the CeraPlus skin barrier\*\*

Greatly Improved 43%  
 Improved 45%  
 Same 9%  
 Worse 3%



**88%**  
**IMPROVEMENT OF SKIN**  
 n = 148

## LONGER WEARTIME

**39%** n = 161 OF PATIENTS WERE ABLE TO WEAR THE CERAPLUS SKIN BARRIER LONGER, ON AVERAGE, THAN THEIR PREVIOUS SKIN BARRIER

# Maintaining Healthy Skin From The Start with the CeraPlus skin barrier


## Before Using

the CeraPlus skin barrier\*

PATIENTS PRESENTING WITH  
**NORMAL SKIN** CONDITION

- Some issues with peristomal skin
- No peristomal skin issue

n = 176

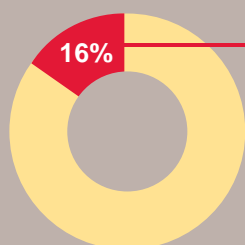

## After Using

the CeraPlus skin barrier\*\*

PERCEIVED IMPROVEMENT  
IN **NORMAL SKIN** CONDITION

Perceived Improved

32%

Remained Normal

64%

Worsened

4%



96%

IMPROVED OR  
REMAINED NORMAL

n = 28



"This product performance is exceptional in terms of peristomal skin maintenance."



## Peristomal Skin Deserves Better



The data produced are not the result of a randomized or controlled study.

\* As reported by their WOC Nurse before use of the CeraPlus skin barrier used as part of overall care plan.

\*\* As reported by their WOC Nurse after the use of the CeraPlus skin barrier as part of overall care plan.

<sup>1</sup> Remois is a technology of Alcare Co., Ltd.

<sup>2</sup> As reported by their WOC Nurse with product used as part of overall care plan.