

Adapt Lubricating Deodorant

General Information

- **Adapt** lubricating deodorant neutralizes odors, eases emptying, and helps prevent pouch static and sticking
- It is not a perfume or masking agent
- Adapt lubricating deodorant should be used in drainable and closed pouching systems only

For Drainable Pouches

1. Apply pouch.
2. Bottle: add approximately 5 mL or 1 teaspoon to bottom of pouch (Figure 1).
Packet: add the complete contents of the packet to bottom of pouch (Figure 2). Close the pouch in the normal manner.
3. Rub to thoroughly coat the inside of the pouch (Figure 3). Repeat Step 2 every time you empty your pouch.

For Closed Pouches

1. Add Adapt lubricating deodorant before removing the release liner on your skin barrier.
2. Bottle: add approximately 5 mL or 1 teaspoon through pouch opening (Figure 4).
Packet: add the complete contents of the packet through pouch opening.
3. Apply pouch.
4. Rub to thoroughly coat the inside of the pouch (Figure 3).



FREQUENTLY ASKED QUESTIONS

Does it matter if the Adapt lubricating deodorant comes in contact with the filter on the pouch?

When coating the inside of the pouch, it is best to avoid the filter.

Do I need to use the Adapt lubricating deodorant every time I empty my pouch?

Yes. For this ostomy accessory to work effectively, it must be added each time your pouch is drained.

Can I just put a few drops in the bottom of my pouch and not rub it against the sides of the pouch?

No, this is not recommended since it will not be effective as a pouch lubricant or as a pouch deodorant. Rubbing the pouch helps to coat the inside of the pouch film.

Isn't it easier to use a cooking spray or oil instead to coat the inside of my pouch?

This is not recommended because cooking oil may cause the seal of your pouch to fail. Also, cooking oil sprays do not eliminate odor.

Adapt Lubricating Deodorant

FREQUENTLY ASKED QUESTIONS

<p>The 8 oz bottle is not convenient to take into a public restroom. Is there a smaller bottle available?</p>	<p>Adapt lubricating deodorant is also available in single-use packets. This is much more discrete and convenient when you are away from home.</p>
<p>It doesn't have a scent. How do I know it is working?</p>	<p>Adapt lubricating deodorant is not a perfume or a fragrance used to mask or cover-up odors. It does not have any odor by itself, but it will neutralize odors in the pouch.</p>
<p>Can the Adapt lubricating deodorant come in contact with the stoma?</p>	<p>Yes, it has been tested and it will not harm your stoma.</p>
<p>What if Adapt lubricating deodorant comes in contact with my skin?</p>	<p>This product is not a skin irritant. It is water-soluble, so it will easily be removed from the skin with a moist cloth. Be sure to wash it off your skin before applying your pouch, so it does not interfere with adhesion.</p>



Adapt lubricating deodorant

Product No 78500
(8 oz / 236 ml bottle)

Product No 78501
(box of 50, 8 mL packets)

For product questions, sampling needs, or detailed clinical questions concerning our products call **1.800.263.7400**.

Routine follow-up with your healthcare professional is recommended.

Prior to using any ostomy products/accessories be sure to read all product inserts and labels.

Hollister Limited
Aurora, Ontario L4G 0Z6
1.800.263.7400

www.hollister.com

The Hollister logo, Adapt and "Healthy skin. Positive outcomes." are trademarks of Hollister Incorporated.
© 2019 Hollister Incorporated. 923061-1217 US-00214